

Instructions for use of CeliaX kit:

CeliaX™

Before taking the sample refrain from eating, drinking, smoking or putting anything in your mouth for 45 minutes



- Before use check that all the details on the sticker match those of the patient
- Paste the sticker on the test tube (without the sticker we won't be able to complete the process)



- **Do not touch the sponge at the tip of the stick**
- **Do not dip the sponge in the blue liquid before completing the test**



Dip the sponge at the tip of the stick in the patient's saliva, and rub her gums on both sides of her cheeks at least ten times on each side.



The stick with the sponge should be screwed off the test tube, turned upside down with the sponge in the test tube with the liquid and closed firmly (see the attached figures)





Mix the content of the test tube gently five times by shaking it

- The tube should be kept in room temperature. Do not cool
- The test tube should be put in the return envelope attached and sent to us by registered mail



For any problems or questions turn to Pronto Diagnostics Labs at telephone no.: 077-5664353